

**PHILIPS**

**RESPIRONICS**

*DreamStation*

Sleep therapy system

Rediscover  
dreams



# Natural. Comfortable. Sleep.

**DreamStation supports long-term patient use while creating new efficiencies for your business**

## **Powerful patient-driven design**

DreamStation was designed with feedback from hundreds of patients to provide a simpler PAP experience to help both new and experienced patients use therapy long-term. A suite of intuitive tools can be customized by healthcare professionals to choose the solution that's right for each patient.

## **Acclimating patients in the early days of therapy**

**EZ-Start** can help patients gradually adapt to their prescribed level of therapy using an innovative pressure acclimation algorithm with automatic, personalized adjustments to PAP pressure over the first 30 days.\*\*

**SmartRamp** allows users to fall asleep to lower pressures than standard ramp mode. The air pressure remains low until the patient experiences an obstructive respiratory event. Then SmartRamp responds with an auto algorithm to resolve the patient's apneas.\*\*

## **Make progress every day**

Each morning patients are greeted with Daily Progress Feedback — a summary of how far they've come to motivate them to stick with their therapy. The intuitive, color display shows a simple trend of their nightly hours of use, followed by a summary of their last 30 days of good nights with more than four hours of use.

## **Sleep therapy success\* at your patients' fingertips**

Bluetooth® is included in every DreamStation device for easy connectivity to the DreamMapper patient self-management tool. DreamMapper helps patients take an active role in their sleep management by providing daily therapy progress, helpful troubleshooting videos, and goal-setting tools. Patients can stay engaged and get the information they need fast, without having to use your staff's time to resolve common issues.

## **An Encore performance**

DreamStation connects to our powerful EncoreAnywhere patient compliance management software. EncoreAnywhere is HIPAA compliant and makes it easy and efficient for sleep professionals to manage patient compliance and therapy.

## **Focus your time where it belongs**

EncoreAnywhere features Adherence Profiler. It's an optional, intuitive tool that helps you identify patients according to their probability of achieving adherence, helping focus your staff's time on the patients that have the greatest likelihood of success.

\*In a retrospective review conducted by Philips Respironics of approximately 15,000 SystemOne patients, patients who used SleepMapper, which has been rebranded to DreamMapper, demonstrated 22% greater adherence to the therapy than patients who did not use SleepMapper. To see a list of compatible DreamMapper devices, go to [www.dreammapper.com/compatible](http://www.dreammapper.com/compatible).

\*\*Not available in all DreamStation PAP models and modes.



My Info

1-Day	7-Day	30-Day
8.0	7.2	6.5
Therapy Hours		



# Creating efficiencies that make a difference

## Simplify device evaluation

Performance Check simplifies in-home device evaluation for easy remote troubleshooting. Designed to reduce the time and frustration involved in device troubleshooting, this robust tool allows you to guide the patient through an easy remote diagnosis, resulting in a simple "Pass/Fail" result to determine if the device needs to be returned for service.

Performance Check can also generate a report in our EncoreAnywhere patient management software tool, allowing you to troubleshoot patient concerns.

**According to our research, 25% of returned devices are found to be operating normally.\*** Performance Check is designed to help reduce the number of normally functioning devices that are returned for service, saving you time and delivery costs.

\*Philips research, 2015, data on file.

\*\*Not available in all DreamStation PAP models and modes.

## Reduce manual adjustments

DreamStation can automatically adjust to changing patient needs, helping reduce costly home visits and letting you focus your attention where it needs to be.



### CPAP-Check

Checks on the user every 30 hours to determine if CPAP pressure is optimal. If not, it automatically adjusts to meet changing patient needs.\*\*



### Auto-Trial

Delivers breath-by-breath auto-CPAP therapy for up to a total 30 days per patient. Transitions to CPAP or CPAP-Check mode at end of Auto-Trial.\*\*



### Opti-Start

Provides optimal starting pressure that can enhance patient comfort and reduce the likelihood of residual events at the beginning of therapy.\*\*

# More cost-effective connectivity options



Only DreamStation offers Bluetooth®, Cellular and Wi-Fi® connectivity options. By providing the most comprehensive suite of connectivity choices among leading PAP brands\*, you can choose the options that are most cost effective. DreamStation's innovative design has the capability to adapt as technologies continue to advance.

#### Making it easy to reach your patients

- Cellular provides daily data to clinicians and one hour after the patient has turned the device off
- Wi-Fi delivers more data to clinicians more often and provides coverage even in weak cellular areas
- Bluetooth is included in every device for easy connectivity with DreamMapper and Alice NightOne

\*Internal assessment of 2015 competitive CPAP data comparing to ResMed Aircurve10/Aircurve10 platform and Fisher & Paykel Icon series platform.

## DreamMapper helps patients take an **active role** in their therapy

With the DreamMapper self-management tool, patients can use the mobile and web-based features to engage with their therapy.



# The performance and appeal patients want

DreamStation offers a sleek, low-profile design. Its vibrant color screen displays daily progress and features easy-to-use menus.

Dreamstation's top-mounted humidifier hose swivel and front-facing display make it easy for patients to operate while lying down or sitting up in bed. And, it's small and light, making it easy to pack for travel.

**30%**  
smaller

The DreamStation PAP is 30% smaller than the Aironse10 PAP.\*\*†

**39%**  
lighter

The DreamStation PAP is 39% lighter than the Aironse10 PAP.\*\*†

**63%**  
quieter

The DreamStation PAP is 63% quieter than ResMed Aironse10.\*\*†

## Hear it from **DreamStation users**

Good-looking piece of equipment! It's modern looking, doesn't look like a medical machine. It's quieter, too.

I liked it as soon as I opened it! I liked the way it was laid out and I liked the size of it — it was small, easy to fit on a table along with a lamp and my phone.

\*Philips internal comparison, 2015, data on file.

\*\*Based on the average change in sound level during simulated patient breathing, across a range of pressure, expiratory relief, and humidifier conditions.

†Does not apply to non-invasive ventilation products



# Insight-driven innovation

## Intuitive menu system

Graphics based menu inspired by smartphone navigation for ease of use

## Ambient light sensor

Automatically adjusts screen brightness for daytime or nighttime viewing




## Integrated Bluetooth

Wirelessly connects with DreamMapper or Alice NightOne portable sleep diagnostic system

## Daily progress feedback

Displays previous 3 nights therapy results to track progress

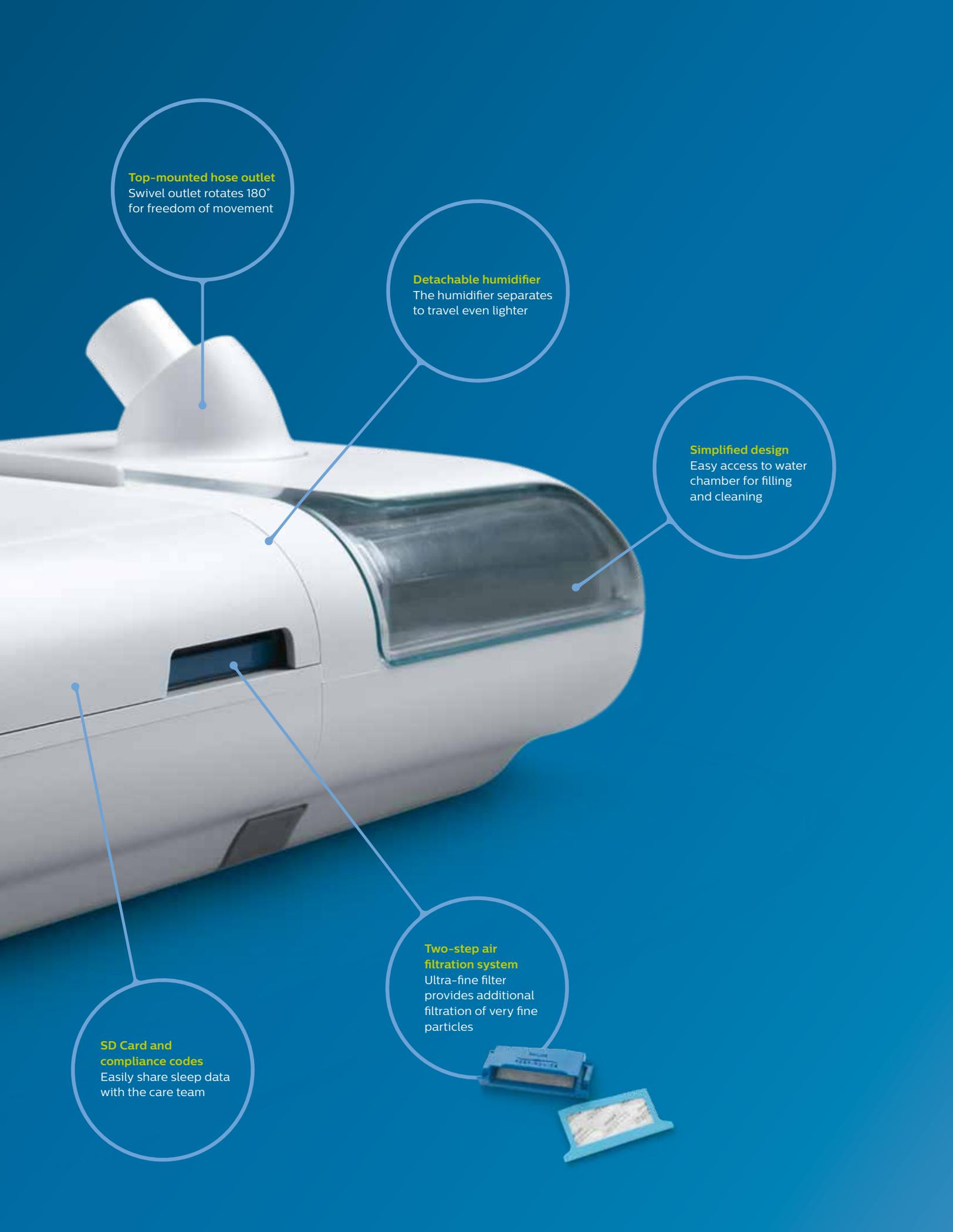

**Top-mounted hose outlet**  
Swivel outlet rotates 180°  
for freedom of movement

**Detachable humidifier**  
The humidifier separates  
to travel even lighter

**Simplified design**  
Easy access to water  
chamber for filling  
and cleaning

**SD Card and  
compliance codes**  
Easily share sleep data  
with the care team

**Two-step air  
filtration system**  
Ultra-fine filter  
provides additional  
filtration of very fine  
particles



# Ordering information

## Sleep therapy

### DreamStation Auto CPAP **Part number**

DreamStation Auto CPAP	DSX500S11
DreamStation Auto CPAP with humidifier	DSX500H11
DreamStation Auto CPAP with humidifier and heated tube	DSX500T11

### DreamStation CPAP Pro **Part number**

DreamStation CPAP Pro	DSX400S11
DreamStation CPAP Pro with humidifier	DSX400H11
DreamStation CPAP Pro with humidifier and heated tube	DSX400T11

### DreamStation CPAP **Part number**

DreamStation CPAP	DSX200S11
DreamStation CPAP with humidifier	DSX200H11
DreamStation CPAP with humidifier and heated tube	DSX200T11

### DreamStation Auto BiPAP **Part number**

DreamStation Auto BiPAP	DSX700S11
DreamStation Auto BiPAP with humidifier	DSX700H11
DreamStation Auto BiPAP with humidifier and heated tube	DSX700T11

### DreamStation BiPAP Pro **Part number**

DreamStation BiPAP Pro	DSX600S11
DreamStation BiPAP Pro with humidifier	DSX600H11
DreamStation BiPAP Pro with humidifier and heated tube	DSX600T11

### DreamStation BiPAP autoSV **Part number**

DreamStation BiPAP autoSV	DSX900S11
DreamStation BiPAP autoSV with humidifier	DSX900H11
DreamStation BiPAP autoSV with humidifier and heated tube	DSX900T11

## Respiratory therapy

### DreamStation BiPAP AVAPS **Part number**

DreamStation BiPAP AVAPS	DSX1130S11
DreamStation BiPAP AVAPS with humidifier	DSX1130H11
DreamStation BiPAP AVAPS with humidifier and heated tube	DSX1130T11

### DreamStation BiPAP S/T **Part number**

DreamStation BiPAP S/T	DSX1030S11
DreamStation BiPAP S/T with humidifier	DSX1030H11
DreamStation BiPAP S/T with humidifier and heated tube	DSX1030T11

## Sleep and Respiratory therapy

### Humidification **Part number**

DreamStation humidifier	DSXH
DreamStation water chamber	1122520
RP-DS Humidifier Dry Box Assembly	1120668
RP-DS Humidifier Flip Lid Seal	1120617
RP-DS Humidifier Dry Box Inlet Seal	1120613

### Tubes **Part number**

15mm Std Tube – DreamStation - RP	PR15
15mm Heated Tube	HT15

### Filters **Part number**

Pollen filter, reusable (1 per pack)	1122446
Ultra-fine filter, disposable (1 per pack)	1122447
Ultra-fine filter, disposable (2 per pack)	1122518
Ultra-fine filter, disposable (6 per pack)	1122519

## Advanced therapies

DreamStation ventilation solutions use clinically proven algorithms that support customized, connected care for patients who require advanced therapy. DreamStation BiPAP autoSV, BiPAP AVAPS, and BiPAP S/T options are designed to offer exceptional comfort, intelligent monitoring, and seamless connectivity between patients and care teams.

Call **1-800-345-6443** or visit [www.philips.com/dreamstation](http://www.philips.com/dreamstation) for more information.




BiPAP autoSV

## Sleep and Respiratory therapy accessories

Power Supplies	Part number
DreamStation 65W power supply	1120136
DreamStation 80W power supply	1118499
Power cord	1005894
PAP battery kit	Part number
PAP Lithium Ion Battery Kit*	1124009
PAP battery kit replacement parts	Part number
Li Ion battery, PAP device cable	1121736
Li Ion Battery, Cable Adapter (50 series adapter, #2)	1127799
Li Ion Battery Case	1126947
DreamStation Travel Case	Part number
Travel case, 65W power supply w/ USB port	1120135
Travel case, 65W power supply w/ USB port, 5/case	1124301

## Sleep and Respiratory therapy accessories

Oximetry	Part number
Link module	1120293
Nonin SpO <sub>2</sub> , assembly	1121694
Reuseable finger sensor (clip)	936
Oximetry finger sensor, adult, flex	953
Flexiwrap, sens tape, adult, 25/pk	954A
Reuseable finger sensor (soft), medium	1062050
Reuseable finger sensor (soft), small	1062051
Modems	Part number
Cellular modem, oximetry capable	100600C
Cellular modem, Non-oximetry	100605C
Wi-Fi accessory, oximetry capable	100700W
Wi-Fi accessory, Non-oximetry	100705W
Travel	Part number
Universal PAP/laptop travel briefcase	1114784
DreamStation replacement carrying case	1121162
Shielded DC cord	1120746
Shielded 12V DC power cord system (includes shielded DC cord and battery adapter cable)	1120747
Mask and tubing bedside organizer	Part number
Bedside organizer	1122135
Bedside organizer, 10/case	1124224

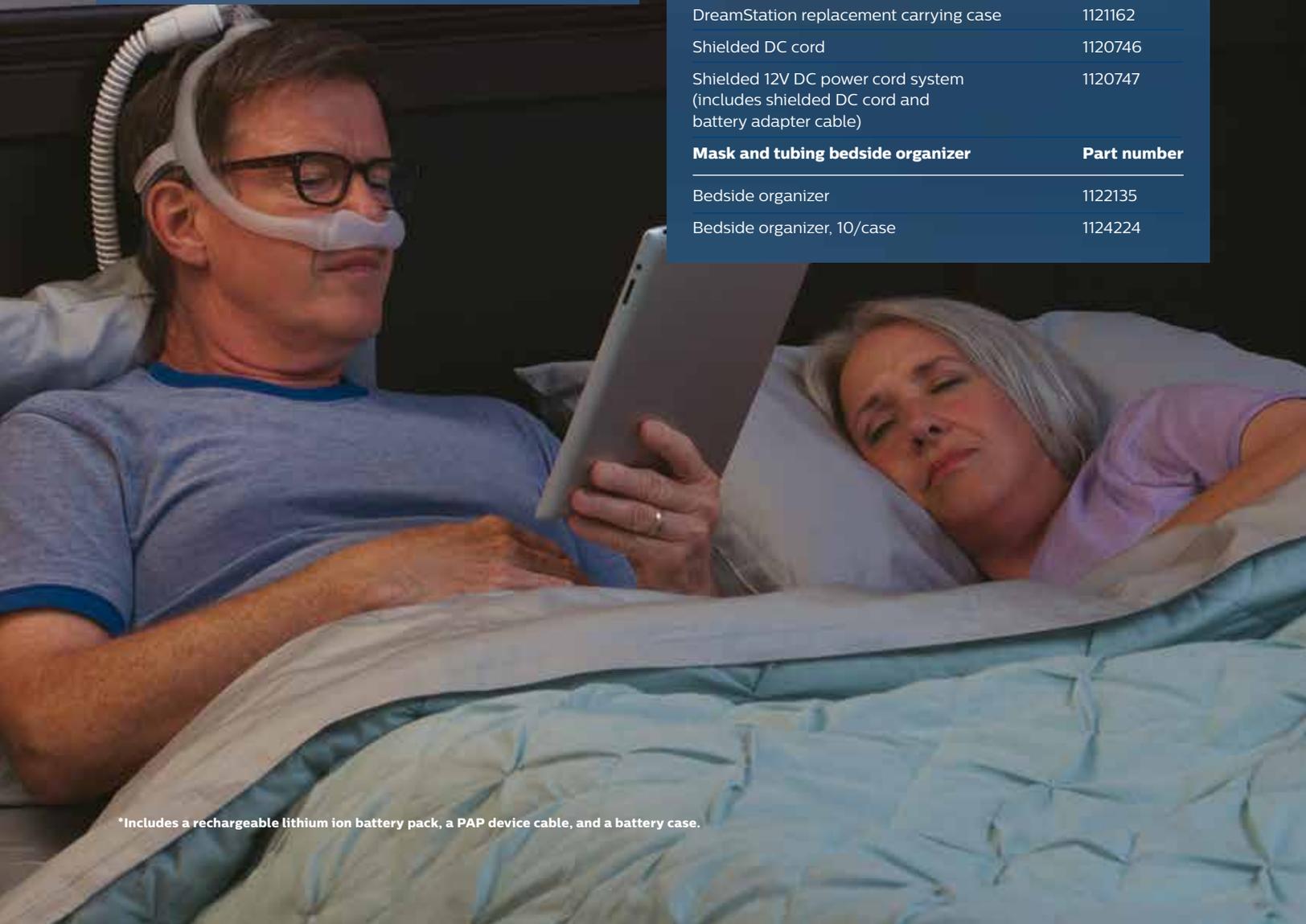

\*Includes a rechargeable lithium ion battery pack, a PAP device cable, and a battery case.



© 2016 Koninklijke Philips N.V. All rights reserved.  
Specifications are subject to change without notice.  
The Bluetooth word mark and logos are registered trademarks owned by  
Bluetooth SIG, Inc. and the use of such marks by Philips Respironics is  
under license. Wi-Fi is a trademark of Wi-Fi Alliance.  
[www.philips.com/respironics](http://www.philips.com/respironics)

Caution: US federal law restricts these devices to  
sale by or on the order of a physician.

RRDPGH KW 8/18/16 MCI 4107682 PN 1125341  
1010 Murry Ridge Lane, Murrysville, PA 15668  
800-345-6443 • 724-387-4000